EXPLANATION OF THE TEST

Panleukopenia due to feline parvovirus (FPV) is one of the most important infectious diseases of cats in the world. The clinical syndromes are gastroenteritis, manifested as diarrhea, emesis, anorexia, and abdominal tenderness. Asan Easy Test[®] FPV is a chromatographic immunoassay kit for rapid and qualitative detection of feline panleukopenia virus antigen in feline feces. The nitrocellulose membrane of the kit is immobilized with FPV-specific antibody in test line and with anti-mouse antibodies in the control line. And also, FPV-specific antibody is conjugated to the colloidal gold particles. This conjugate is placed on a polyester or glass fiber as conjugate pad. When the sample is dropped into the sample well on the device, the solubilized conjugate migrates with the sample by passive diffusion and both the conjugate and sample come into contact with the antibody that immobilized onto the nitrocellulose. If the sample contains FPV antigens, the result is visible as red line within 10 minutes in the test line on the membrane. The solution continues to migrate to encounter a control reagent that binds a control conjugate, thereby producing another red control line.

MATERIALS PROVIDED

Asan Easy Test[®] FPV contains the following components (10 Test/Kit)

1. Test device individually foil-pouched with a desiccant : 10 units
2. Assay solution in tube (1 mL/tube) : 10 units
3. Disposable dropper : 10 units
4. Sterilized swab : 10 units
5. Instruction manual for use : 1 sheet

PRECAUTIONS


1. The presence of humidity may decrease the stability of the reagents. Thus, please carry out the test immediately after removing the device from the foil pouch.
2. Do not use the kit after the expiration date and do not freeze the kit.
3. For veterinary use only.
4. Wear protective gloves while handling samples and wash hands thoroughly after the test.
5. Dispose all the specimens and kits properly after test, in accordance with GLP.
6. Never use reagents from another kit.
7. Discard the Assay Solution if it is contaminated with bacteria or mold.

SPECIMEN COLLECTION AND STORAGE

Specimen of this test is feline fecal material (stool). And, it should be obtained and handled by standard methods for the sample collection. The feces specimens must be tested as soon as early they are collected. If necessary, they may be stored at 2-8°C for up to 48 hours. If longer storage is required, the sample may be frozen at -20°C.


TEST PROCEDURE

1. Place all specimens, test devices, and allow them to room temperature prior to testing (15~30 min).
2. Remove the test device from the sealed pouch, and place it on a clean and plain place.
3. Collect the feces specimens at 4 different points or directly take the specimens from the canine colon.
4. Insert and mix the swab with assay solution in a tube until the specimens have been dissolved. Then, discard the swab and repeat procedure for each sample.
5. Add 3~4 drops (approx. 100 μL) of the solution taken by the disposable dropper into the sample well.
6. Within 10 minutes, interpret the test result(s). Do not read after 10 minutes.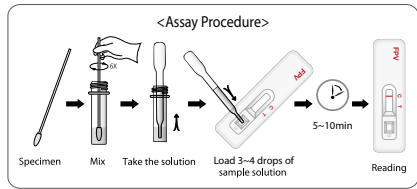

INTERPRETATION OF THE TEST

1. Negative result : ONLY one band in the control line (C).
2. Positive result : Two bands are appeared in the test line (T) and control line (C)
3. Invalid result : If at 10 minutes, the red color band does not appear in the control line (C), even if any shade of a pink-to-red test line (T) appears, the result is considered invalid. If the test is invalid, a new test should be performed with a new sample and a new test device.

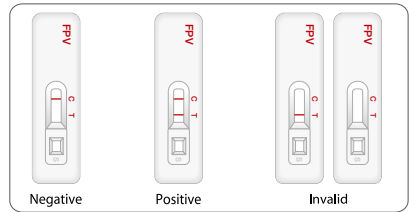STORAGE & EXPIRATION

1. Asan Easy Test FPV kit should be stored between 1 to 30°C (33.8-86 °F).
2. Expiration date of this kit is 24 months after its manufacture date.

PERFORMANCE CHARACTERISTIC

Asan Easy Test[®] FPV is designed for primary screening test of feline parvovirus antigen. This kit can provide fast and easy way to get a result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test results, and veterinarian's view, collectively.

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