INTENDED USE

Asan Easy Test[®] FCV Ag is a chromatographic immunoassay kit for rapid and qualitative detection of calicivirus antigen in feline secretions of eye mucus, nasal discharge and saliva. A nitrocellulose membrane is immobilized with FCV antibody in the test line, respectively. FCV antibody is conjugated to colloidal gold particles. This conjugate is placed on a polyester pad as a conjugate pad. This test is aimed to the detection of FCV antigen either in saliva and nasal secretions. When the device is loaded with assay buffer containing specimens, the solubilized conjugate migrates with the sample by passive diffusion and the FCV antibody-gold conjugate and FCV antigen in the sample are firstly reacted. Then, the complex comes into contact with control line that it adsorbed onto the nitrocellulose. The result is visible within 10 minutes in the form of a red or purple line.

MATERIALS PROVIDED

Asan Easy Test[®] FCV Ag contains the following components:

1. Test device individually foil-pouched with a desiccant : 10units
2. Assay solution in tube(0.5ml/tube) : 10units
3. Disposable dropper : 10units
4. Sterilized swab : 10units
5. Instruction manual for use : 1sheet

PRECAUTIONS


1. The presence of humidity may decrease the stability of the reagents.
2. Do not use the kit after the expiration date and do not freeze
3. For veterinary use only.
4. Wear protective gloves while handling samples and wash hands thoroughly after the test
5. Dispose all the specimens and kits properly after test, in accordance with GLP.
6. Never use reagents from another kit.
7. Discard the assay solution if it contaminated with bacteria or mold.

SPECIMEN COLLECTION AND STORAGE

1. The test should be performed using feline secretions of eye mucus(in the conjunctiva), nasal discharge or saliva.
2. Specimen to be tested should be obtained and handled by standard methods for the collection of body secretions.
3. After collecting the specimen using swab, all the specimens should be immediately extracted and tested. If necessary, they may be stored at 2~8 °C for up to 48hours or at -20 °C for longer periods.


TEST PROCEDURE

1. Place all specimens, test devices, and allow them to room temperature prior to testing (15~30 min).
2. Remove the test device from the sealed pouch, and place it on a clean and plain place.
3. Take a portion of secretions of saliva and nasal, using a sterilized swab.
4. Insert and mix the swab with assay solution in a tube until the specimens has been dissolved. Then, discard the swab and repeat procedure for each sample.
5. Add 3drops (approx. 100μl) of the assay solution into the sample well(s).
6. Within 10 minutes, interpret the test result. Do not read the result after 10minutes.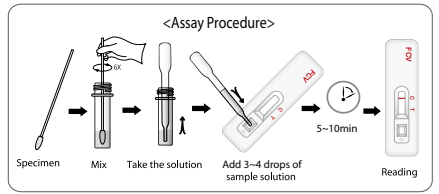

INTERPRETATION OF THE TEST

1. Negative result : ONLY one band in the control line (C).
2. Positive result : Two bands appear in the test line (T) and control line(C).
3. Invalid result: If at 10 minutes, the red color band does not appear in the control line (C), even if any shade of a pink-to-red test line(T) appears, the result is considered invalid. If the test is invalid, a new test should be performed with a new sample and a new test device.

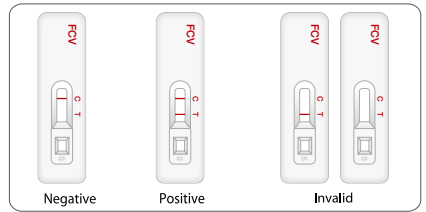STORAGE & EXPIRATION

1. Asan Easy Test[®] FCV Ag kit should be stored between 1 to 30°C (33.8~86 °F).
2. Expiration date of this kit is 24 months after its manufacture date.

LIMITATION OF THE TEST

Asan Easy Test[®] FCV Ag is designed for primary screening test of Feline Calicivirus antigen. This kit can provide fast and easy way to get a result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test results, and veterinarian's view, collectively.

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