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http://www.asanpharm.com

Asan EasyTest DTM

Rapid Detection Medium for Pathogenic Dermatophytes

● APPLICATION

Asan EasyTest DTM is rapid testing medium for pathogenic dermatophytes from the skin disease or dermatitis.

● PRINCIPLE OF THE TEST

The principle of this test is the color change of the medium. If there are pathogenic dermatophytes in the medium, the orange color of the medium is changed to red color.

● MATERIALS PROVIDED

Asan EasyTest DTM: the transparent tube containing the orange color medium

● TEST PROCEDURE

1. If the disease skin is contaminated, please clean the skin using non-anti-microbial shampoo and then dry the site.
2. Identify and select the pathogenic skin or horny substance. Wood lamp positive or damaged wool is also one of good specimen for this test.
3. Take out the wool from the pathogenic site using forceps. It does not need lots of wool for this test.
4. Open the cap of Asan EasyTest DTM and then carefully inoculate the wool or horny substance onto the medium.
5. Close the cap but do not tightly close (ventilation is required). And then, mark the name, number, date, and other information of the sample.
6. Culture the inoculated Asan EasyTest DTM in the shade condition at room temperature.
7. Check and observe the color of medium at every 24 hours.

● PACKAGE

- 10 Tests/kit or others

● STORAGE & EXPIRATION

1. Asan Easy Test DTM should be stored at between 2~8℃ (35.6~46.4°F)
2. Expiration date of this kit is 12 months after its manufacture date.

● CLAIMS

This product is manufactured under our strict quality control. However, if some claims occurs, please contact us (Tel +82-32-376-5990).

REF Code No.: 22331, 22335
Document Code: ADTM00-D0K00

ISO13485:2003

Manufactured & Sold by

ASAN PHARMACEUTICAL CO., LTD

Head Office: 163, Yeongcheon-ro, Dongtan-myeon,

Hwaseong-si, Gyeonggi-do 445-813, Korea

Factory: 122-26, Giupdanji-ro, Gongdo-eup,

Anseong-si, Gyeonggi-do, 456-821, Korea

Tel: +82-31-376-5990~2

Fax: +82-31-376-5993

http://www.asanpharm.com