



# Asan Easy Test<sup>®</sup> H.pylori

Diagnostic Kit for Helicobacter pylori Antibodies Detection

IVD

Immunochromatography

## EXPLANATION OF THE TEST

Asan Easy Test<sup>®</sup> H. pylori is an immunochromatographic assay for the rapid quantitative detection of human anti-H.pylori antibodies in the serum or plasma. The Asan EasyTest<sup>®</sup> H. pylori kit contains the membrane, pre-coated with H. pylori specific antigens on the test line region. During the test, the sample is allowed to react with the colloidal gold-H.pylori-specific antigens conjugate, which has been pre dried on the test strip. This mixture then moves upward on the membrane by capillary action. For positive results, a visible line with high sensitivity and specificity as forming antibody-antigen-gold complex appears in the test line region of the membrane.

## MATERIALS PROVIDED

Asan Easy Test<sup>®</sup> H. pylori contains following items to perform the assay.

1. Test device in aluminium pouch with a desiccant.
2. Assay solution.
3. Instruction manual for use.

## PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. The test kit should remain in the sealed pouch until ready for use.
3. The test kit is sensitive to humidity and to heat.
4. Do not smoke, eat or drink in areas where specimens or kit component are handled.
5. All specimens and reagents should be considered potentially hazardous and handled in the same manner as an infection agent.
6. Wear protective gloves while handling samples and wash hands thoroughly after the assay is complete.
7. Avoid any contact with the eyes, broken skin or mucous membranes.
8. The test device and all materials should be discarded in a proper biohazard container after testing.


## SPECIMEN COLLECTION AND STORAGE

1. The test should be performed using human serum or plasma.
2. If specimens are not immediately tested, they should be refrigerated at 2~8 °C . For storage periods more than three days, freezing is recommended.
3. Specimens containing precipitate may yield inconsistent test results. Such specimens must be clarified prior to assay.


## TEST PROCEDURE

1. All materials should be equilibrated to room temperature before performing test.
2. Remove the device from its protective pouch.
3. Add 20 µl of serum or plasma into the sample well(S).
4. Dispense 3 drops of assay solution into the sample well(S).
5. Interpret test results within 10 minutes. Do not interpret after 20 minutes.

**(CAUTION)** : The above interpretation time is based on reading the test results at room temperature of 15~30 °C . If your room temperature is significantly lower than 15 °C , then the interpretation time should be properly increased.)

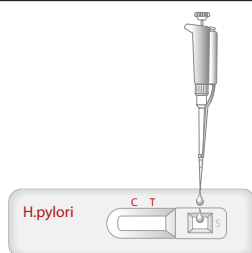

Add 20 µl of serum or plasma

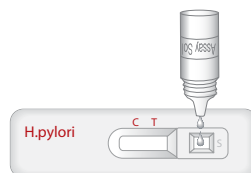

3drops of assay solution



10min



Reading

## INTERPRETATION OF THE TEST

1. A color band will appear in the upper section ("C" zone) of the window to show that the test is working properly. This band is a control band.
2. The down section ("T" zone) of the window indicates the test results, test band. If another color band appears in the down section of the window, this band is a test band.

### A. NEGATIVE RESULTS:

The presence of only one purple color band("C" band) within the result window indicates a negative result .



### B. POSITIVE RESULTS:

The presence of two color bands ("T" band and "C" band) within the result window regardless of which band appears first indicates a positive result .




### C. INVALID RESULTS:

If no band is visible within the window after performing the test, the result is considered invalid . Some causes of invalid results are: not following the directions correctly or the test may have deteriorated beyond the expiration date. It is recommended that the specimen be re-tested using a new test kit.



## STORAGE & EXPIRATION

1. Asan Easy Test<sup>®</sup> H. pylori should be stored at 1 ~ 30 °C (33.8 ~ 86 °F).
2. Expiration date of this kit is 24 months after its manufacture date.

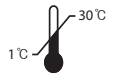

## PERFORMANCE CHARACTERISTICS

This clinical test was performed using a total of 143 specimens. Each specimen was tested with Asan Easy Test<sup>®</sup> H.pylori and commercial kit. The results are summarized in the following tables.

n= 143		Commercial Kit		Total
		Positive	Negative	
Asan Easy Test <sup>®</sup> H.pylori	Positive	62	7	69
	Negative	3	71	74
Total		65	78	143

※ Relative Sensitivity : 95.4 %, Relative Specificity : 91 %

※ Detection limit : 0.76 s/CO

## LIMITATIONS OF THE TEST

Other clinically available tests are required if questionable results are obtained. As all diagnostic tests, a definitive clinical diagnosis should not be based on the results of a single test, but be made by the physician after all clinical and laboratory findings have been evaluated.



Manufactured & Sold by  
**ASAN PHARMACEUTICAL CO., LTD**  
Head Office1: 163, Yeongcheon-ro, Hwaseong-si, Gyeonggi-do 18462, Korea  
Head Office2: 122-26, Gieopdanji-ro, Gongdo-eup, Anseong-si, Gyeonggi-do, 17551, Korea

Tel: +82-31-376-5990~2  
Fax: +82-31-376-5993  
<http://www.asanpharm.com>  
Code No.: 22191, 22196  
**REF** Document Code: AHHPY0-S0E00