




Asan Easy Test® Noro

A fast, easy, reliable test that provides results within 15 minutes.

- ☑ **Easy to use**
- ☑ **Fast result (within 15 minutes)**
- ☑ **A preliminary screening test for early stage and mild symptom patients**

Symptoms of Norovirus

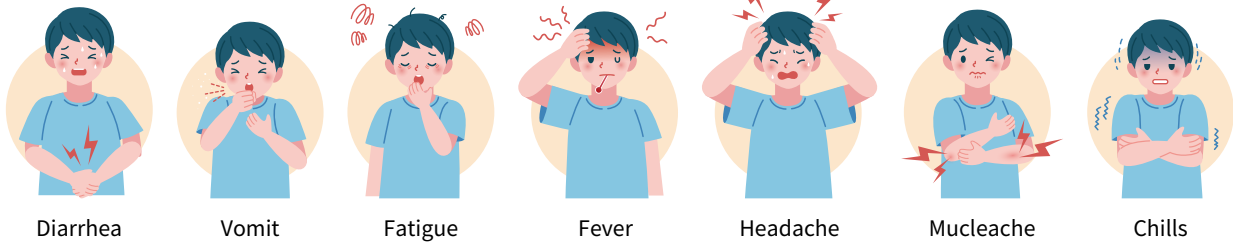

Test procedure

- 1) Put the patient's fecal residual sample (0.1g) into a tube containing the extraction buffer (1.0 mL) and rub it against the tube wall, turning it at least 8 to 10 times to ensure sufficient extraction.
- 2) After extraction, the swab is taken out while turning it along the wall of the buffer tube. During this process, squeeze out the extract solution on the swabs.
- 3) Used swabs are classified as infectious waste and disposed of safely.
- 4) Remove the device from the aluminum pouch and place it on a flat surface. After combining the sample extraction buffer tube with the dropper filter tip to the sample well of the test device, 4-5 drops (120~150 μl) are added to react.
- 5) If the control line appears red after 15 minutes, read the result.
- 6) As the reaction time increases, the degree of color development of the control line and the test line can become stronger. Therefore, it is possible to obtain more accurate results by always making the determination at a certain time after the start of the reaction.

Order Information

Catalog Number	Description	Packaging
AM2371-K	Asan Easy Test [®] Noro	25 Test / Kit

Packaging may vary according to order quantities.

Contact

✉ global@asanpharm.co.kr
☎ +82-31-376-5990



SCAN HERE!